

- ACTIONS**



Circle X: Close PACS and dictation

Magnifying glass: Open in PACS only.

Microphone: Open in Dictation only.

mic/magnifying glass: Open study in Dictation & PACS.

- Assignment**




Down arrow: Open in PACS only, but keeps locked in zVision even though no longer viewing images.


Check Mark: Assign Exam.

Lock: Lock Exam to yourself


- My Reading Queue**

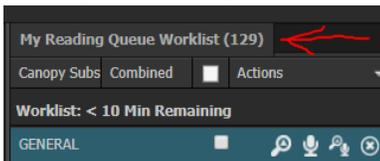

- (Top right of screen) Will take you back to your assigned shift(my reading queue).
- To see the worklists for your selected shift

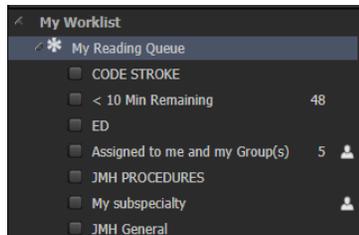


- Manual Worklist Refresh**

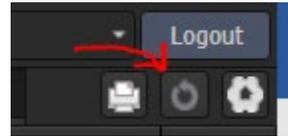

(Top Left of Screen.)


- Quick Search (Bottom left)**

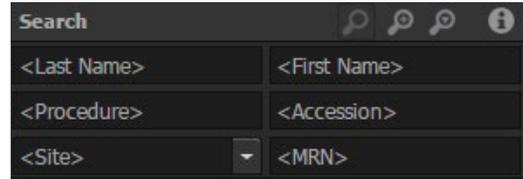

- Last and First Name.** These boxes have a 3 minimum limit.
- Accession number** prepend with % .
- MRN** prepend with % *.

- Auto-Next (A-N)**

- Select your worklist
- Click the start button

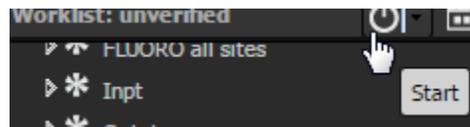


- Auto next buttons.
 - Balloon:** Read Next
 - Curved Arrow:** Skip



- Stop:**



- Add an exam:**

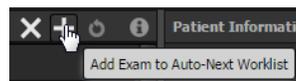


- Tech QA Note**




For positive/constructive tech feedback. Notes are sent to hospital administration.

- Communication Note**

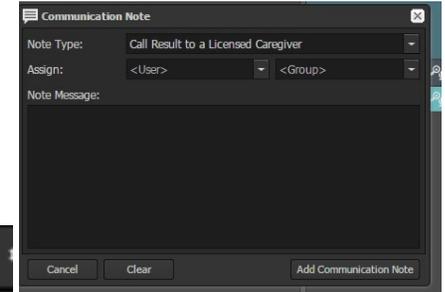

- Select note type
- Type your note message
- Add communication Note
- This will appear on the appropriate person's worklist

- Change/Set Shift**

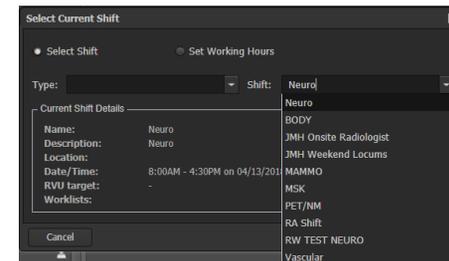


Click save once you choose shift.


Your new shift will be "my reading queue".

- **My History**

1. The **History** box is located in the bottom left corner.

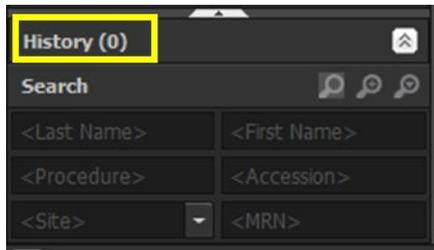


2. Click and drag the box up to expand the list

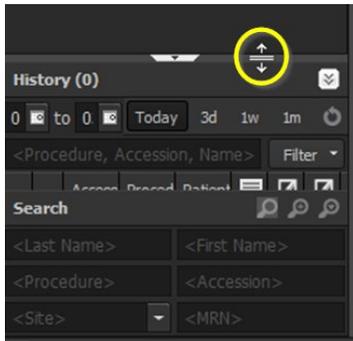

3. Choose one of the below options to view your history.
 - a. **Enter date range**
 - b. **Today** = today's history
 - c. **1w** = last 7 days
 - d. **Custom** = date range



4. Search for your patient.

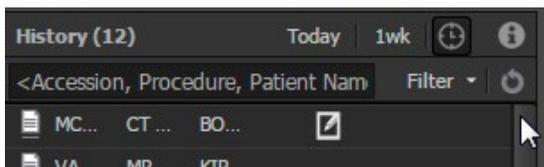


5. Double click on the patient to select it. This will open the patient information window.

- **Edit Columns in Worklist**

1. Put the mouse cursor over the white triangle on one of the columns and left click.



2. Click on columns menu and select/deselect column heading by clicking checkmark.

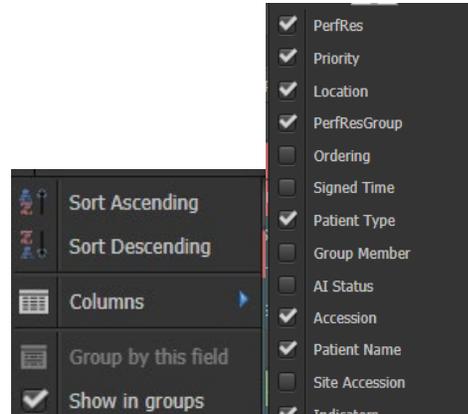

3. Adjust columns
 - a. Left click and drag column to the desired location.
 - b. Here the patient name column will be moved where the blue arrow is shown.

